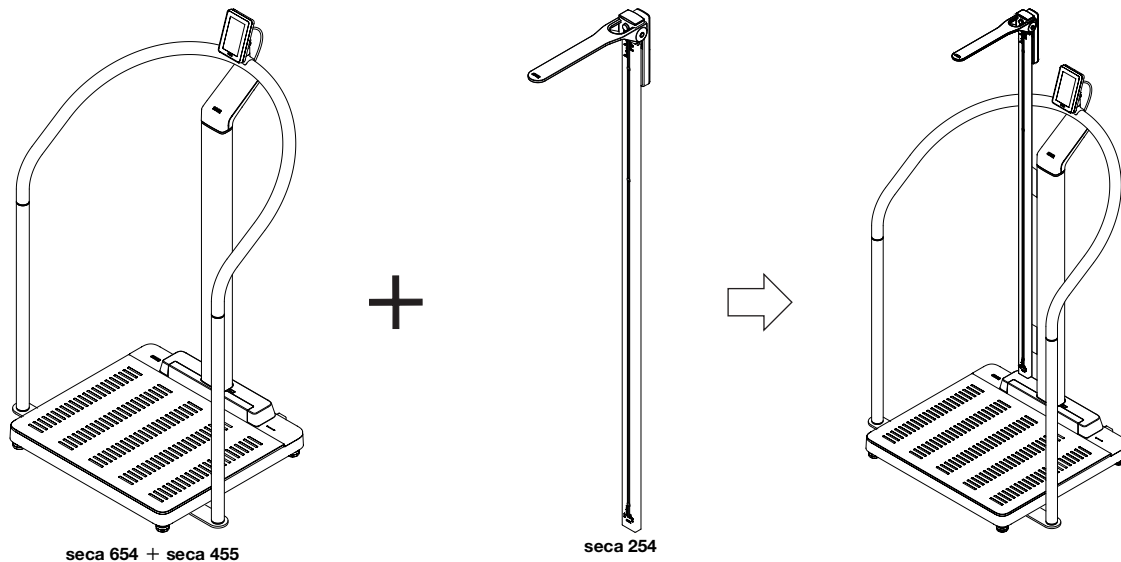
Mounting instructions

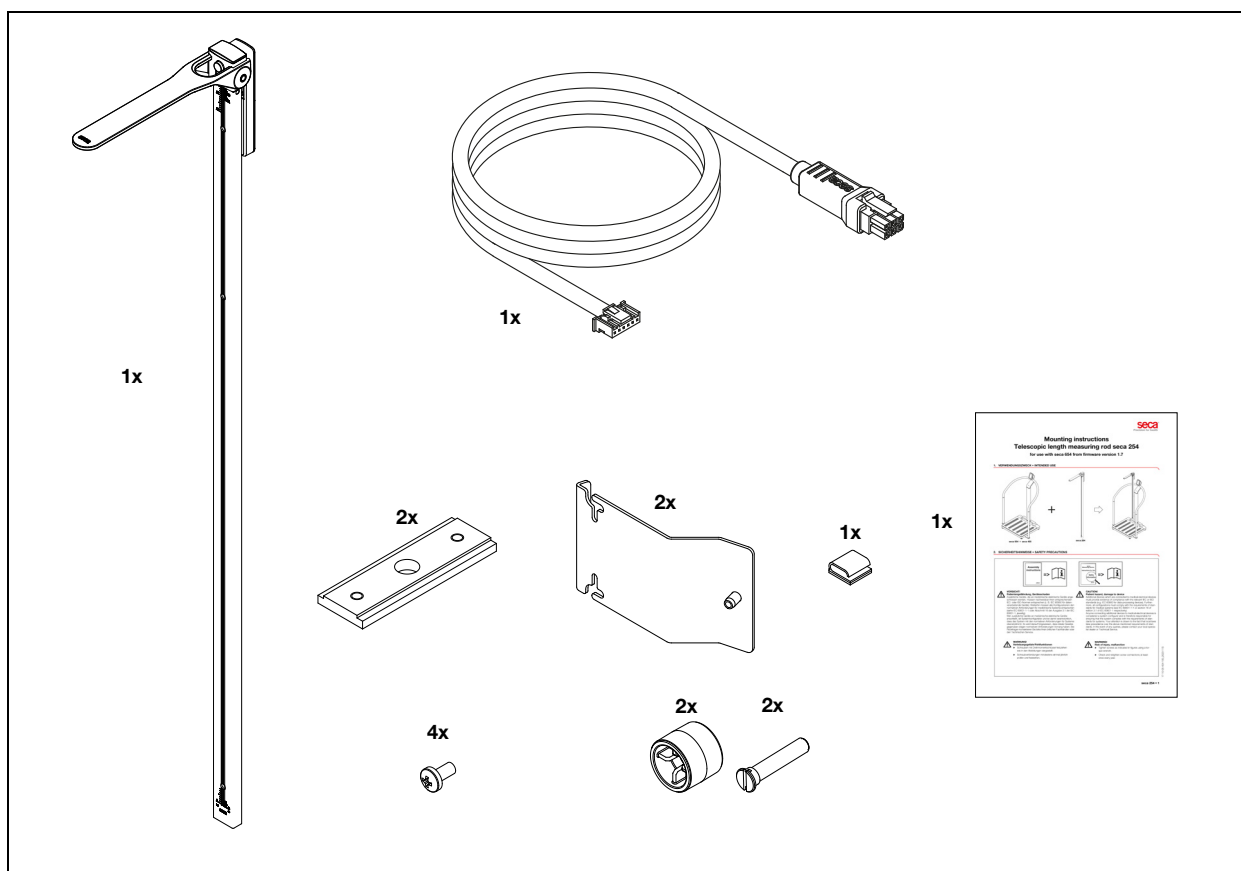
Telescopic length measuring rod seca 254

for use with seca 654 from firmware version 1.7

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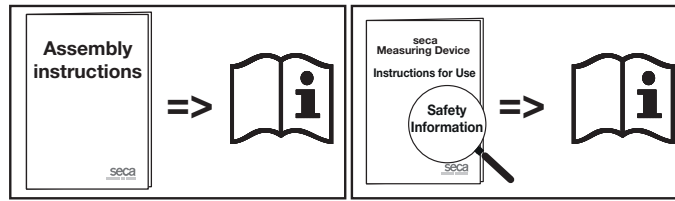
CAUTION!

Patient hazard, damage to device

Additional devices which are connected to medical electrical devices must provide evidence of compliance with the relevant IEC or ISO standards (e.g. IEC 60950 for data-processing devices). Furthermore, all configurations must comply with the requirements of standards for medical systems (see IEC 60601-1-1 or section 16 of edition 3.1 of IEC 60601-1 respectively).

Anyone connecting additional devices to medical electrical devices is considered a system configurator and is therefore responsible for ensuring that the system complies with the requirements of standards for systems. Your attention is drawn to the fact that local laws take precedence over the above-mentioned requirements of standards. In the event of any queries, please contact your local specialist dealer or Technical Service.

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WARNING!

Risk of injury, malfunction

- ▶ Tighten screws as indicated in figures using a torque wrench.
- ▶ Check and retighten screw connections at least once every year.

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CAUTION!

Assembly error/malfunction

The measuring rod **seca 254** is compatible to the battery pack **seca 453**.

- ▶ When mounting both devices make sure to mount the battery pack **seca 453** first and the measuring **seca 254** second.

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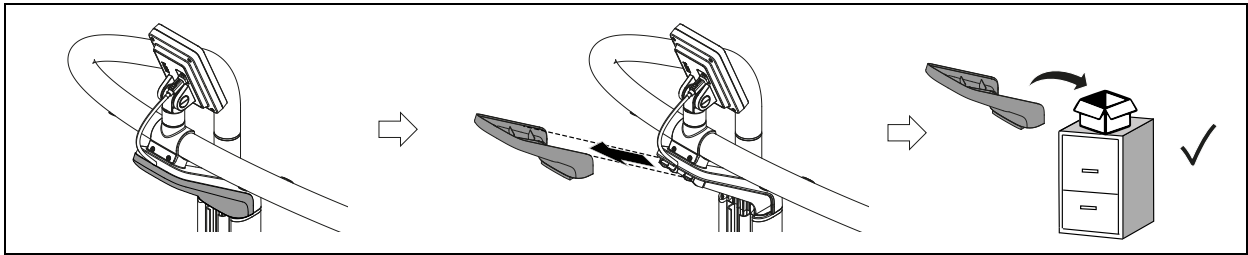
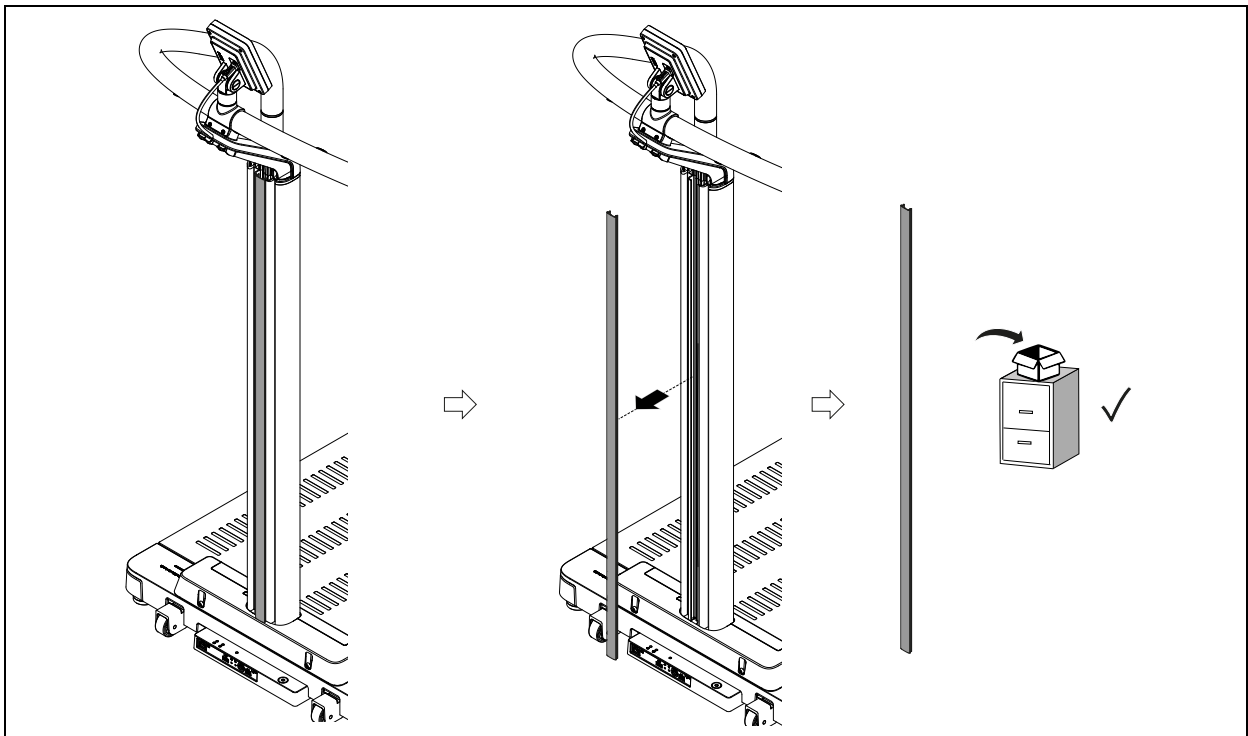
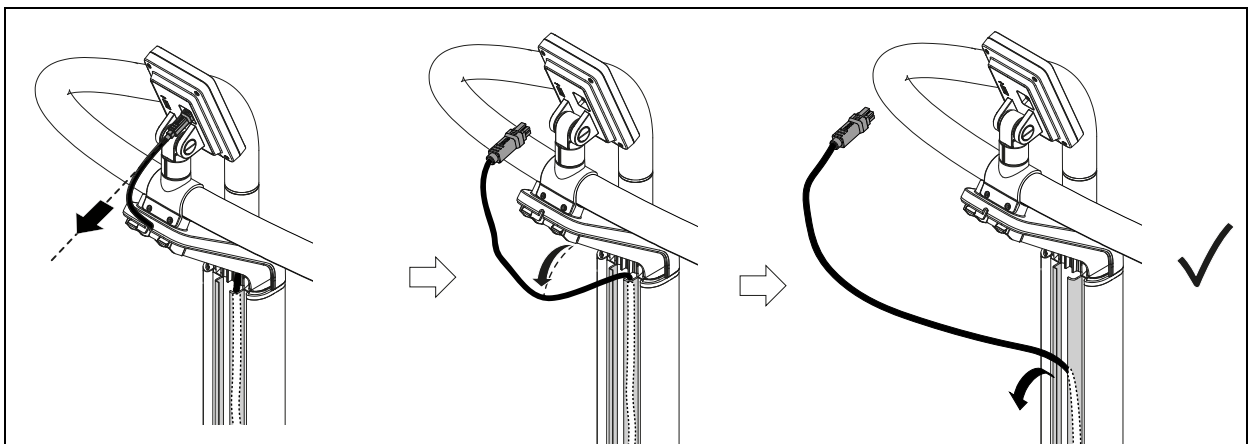
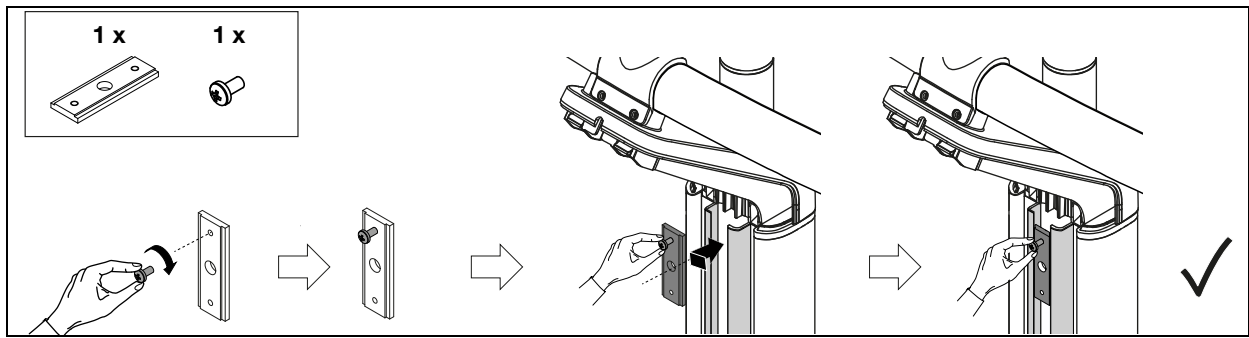
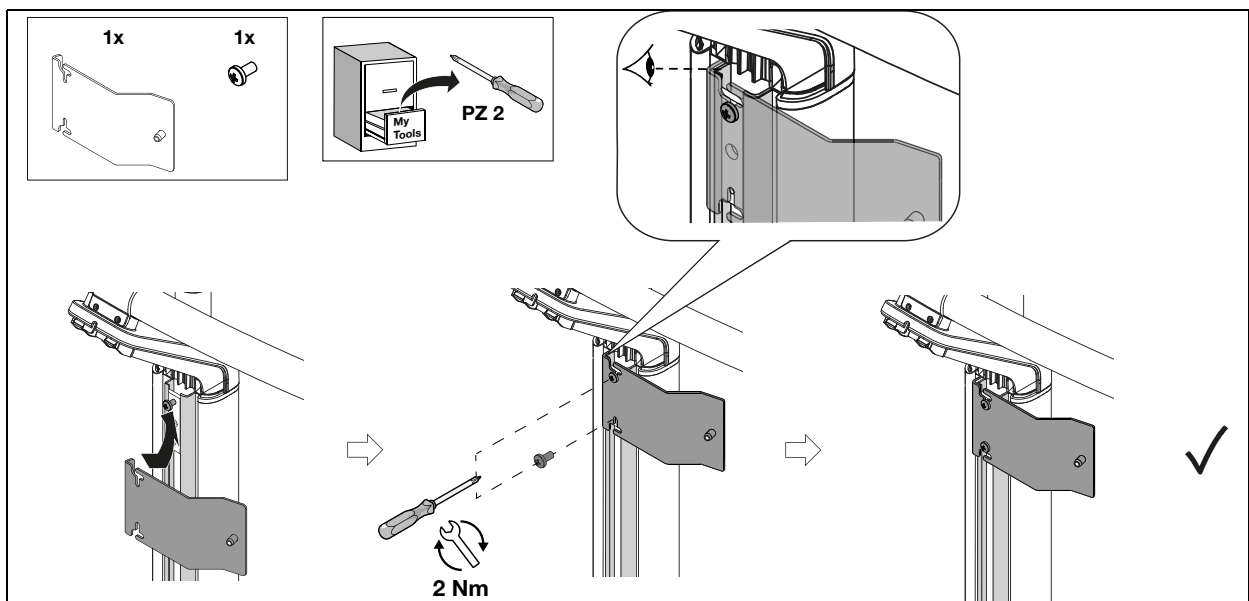
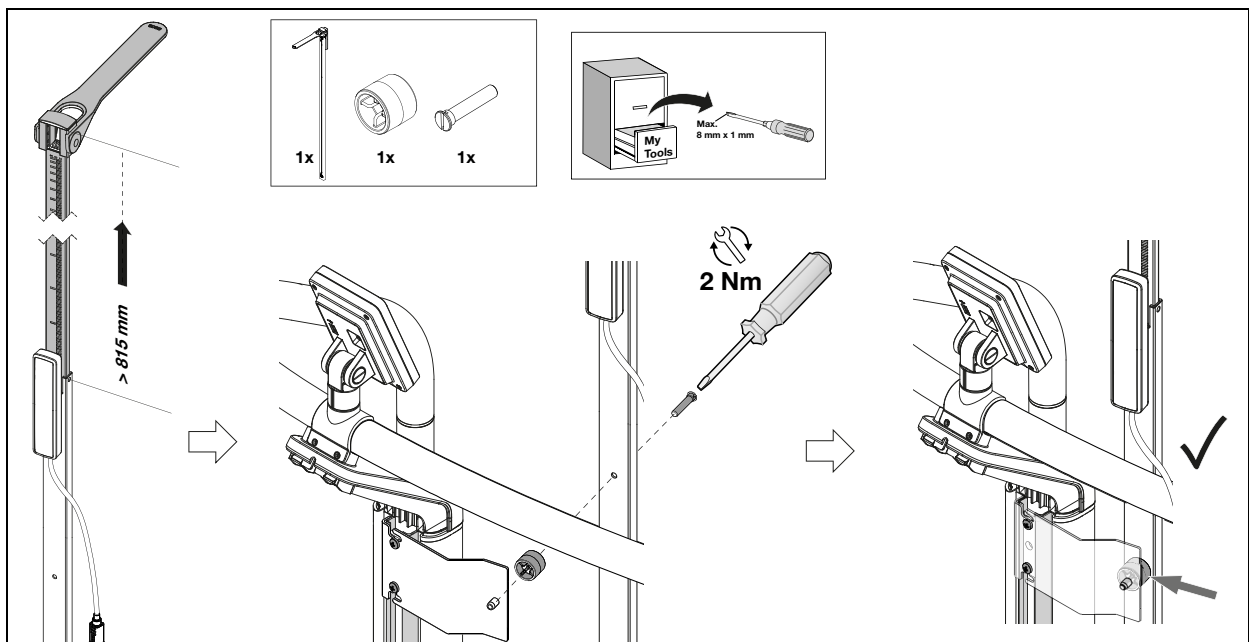
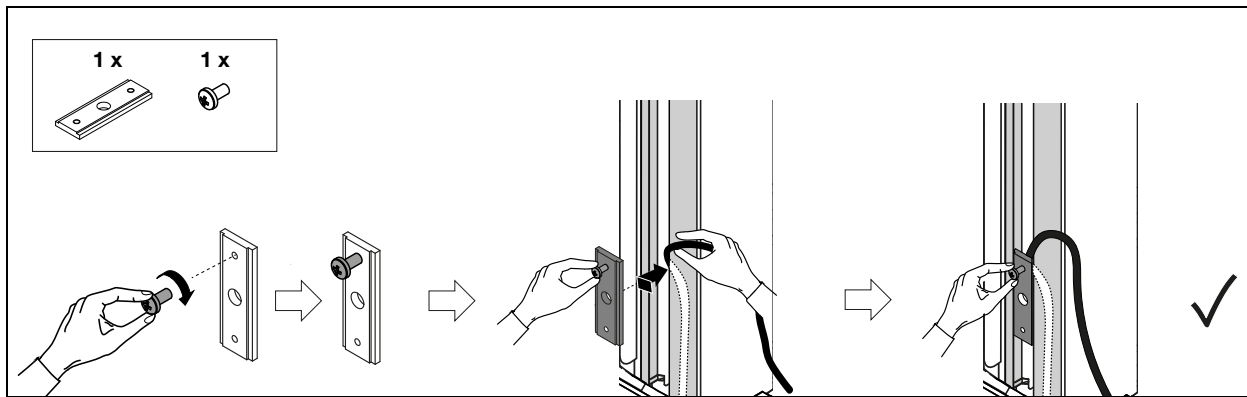
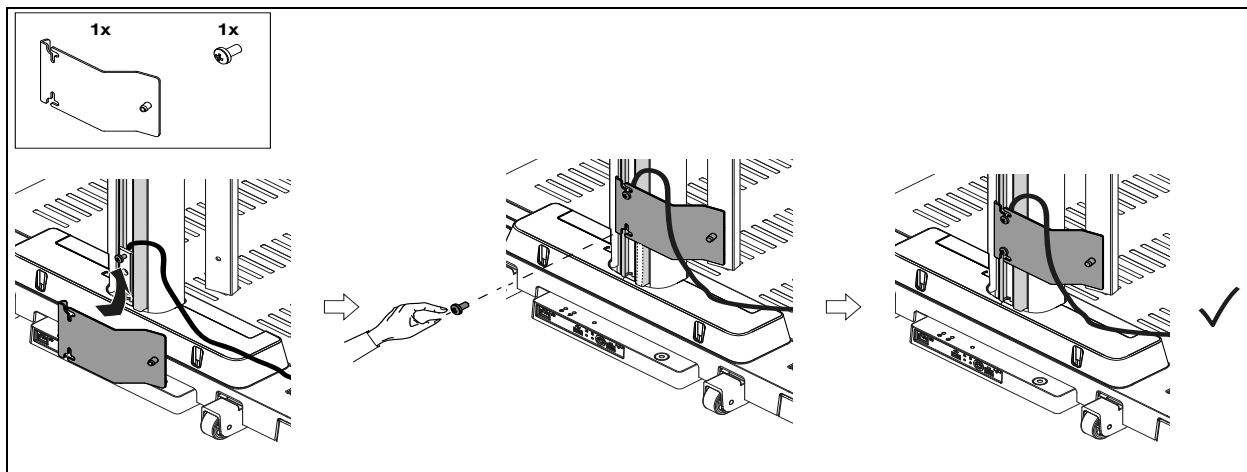
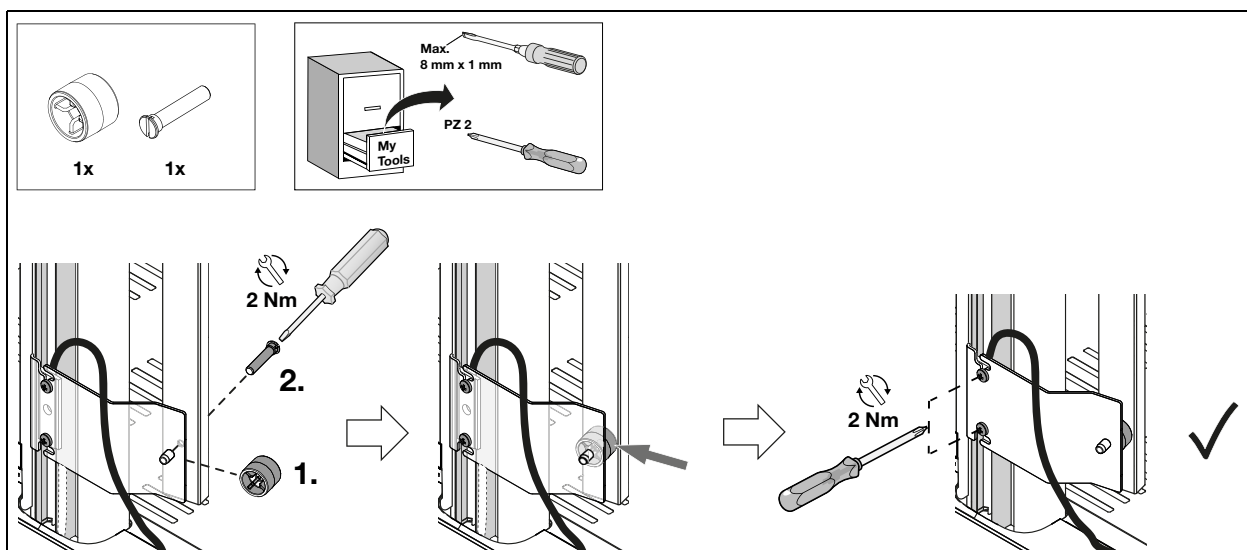
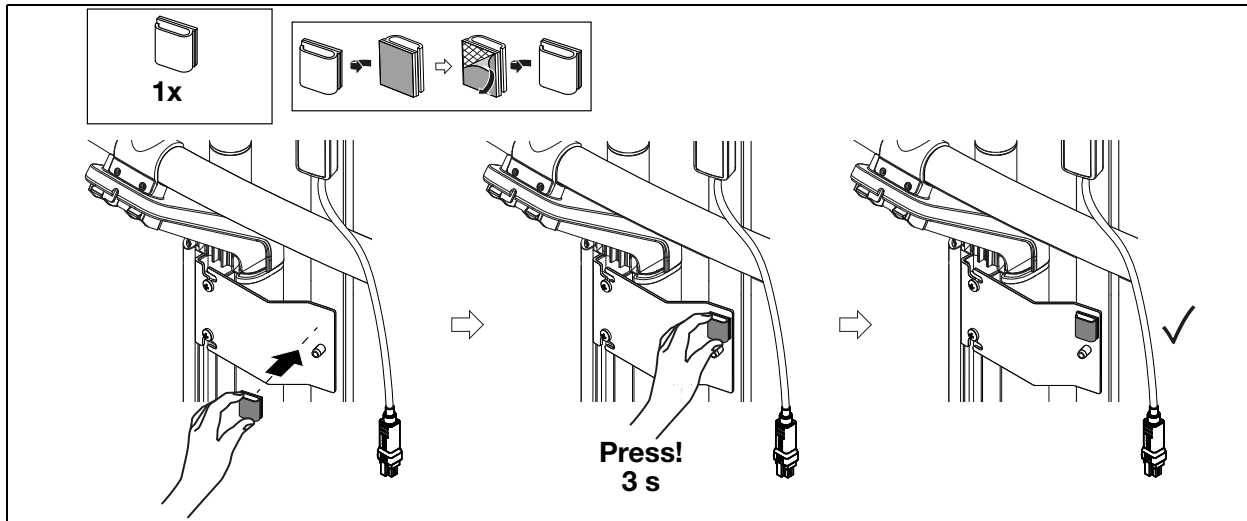
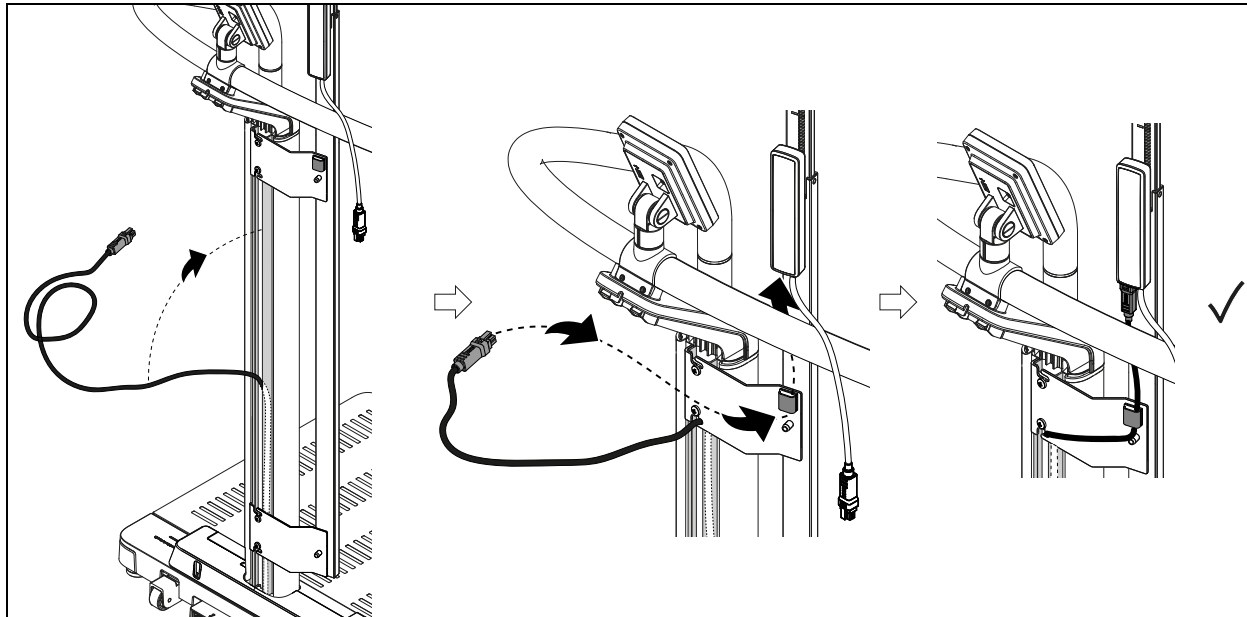
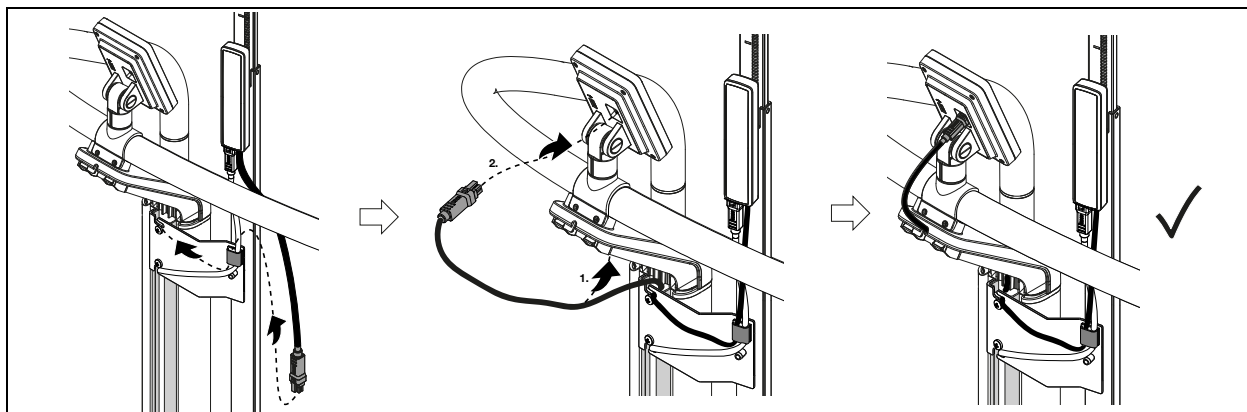
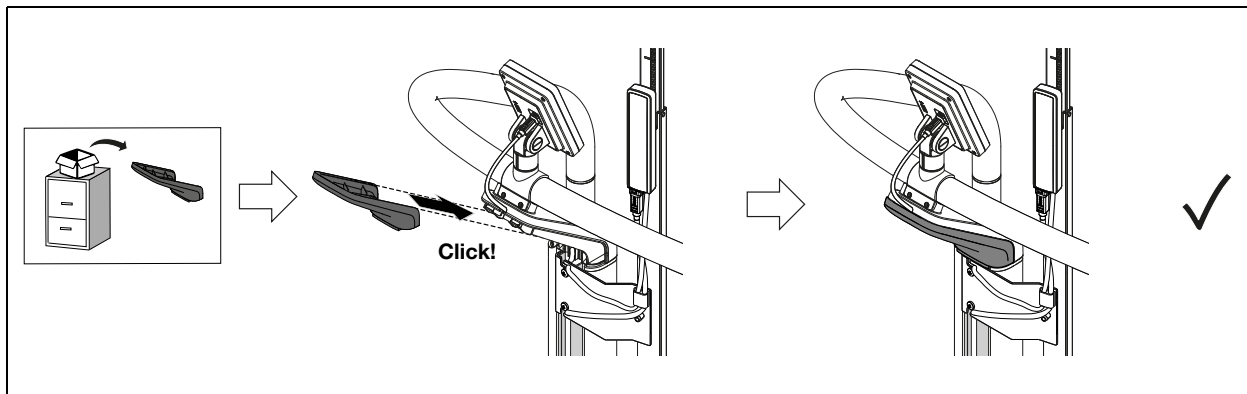
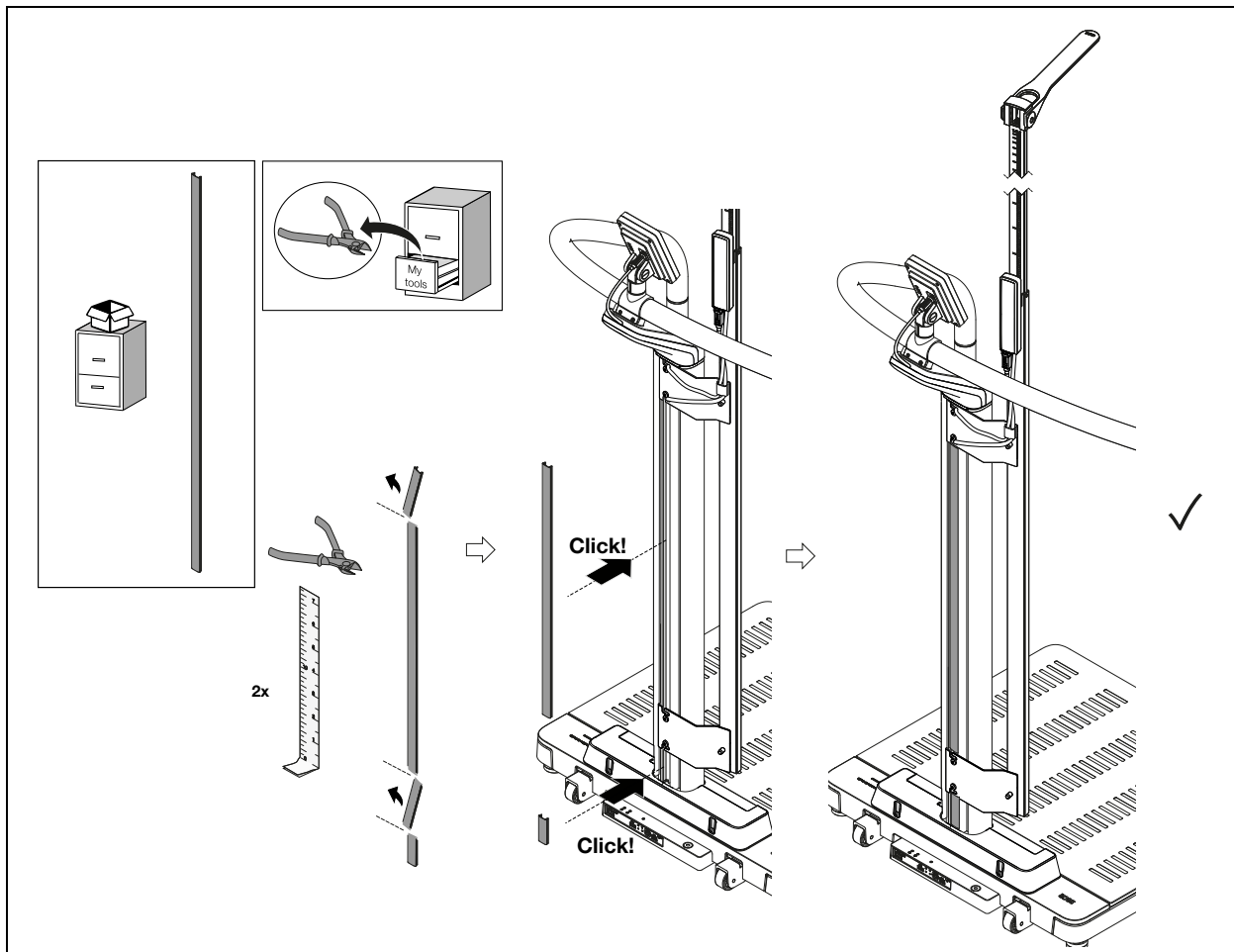
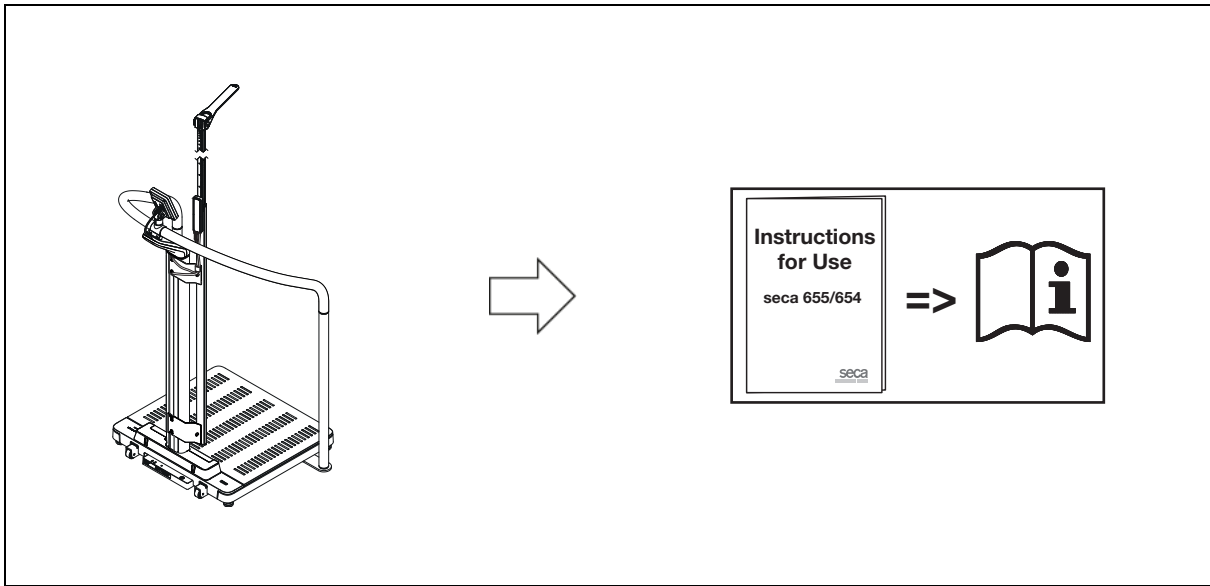
Medical Measuring
Systems and Scales
since 1840

seca gmbh & co. kg
Hammer Steindamm 3-25
22089 Hamburg
Germany

T +49 40 20 00 00 0
F +49 40 20 00 00 50
E info@seca.com

seca operates worldwide
with headquarters in
Germany and branches in:
seca france
seca united kingdom
seca north america
seca schweiz
seca zhong guo

seca nihon
seca mexico
seca austria
seca polska
seca middle east
seca suomi
seca américa latina
seca asia pacific

seca danmark
seca benelux
seca lietuva
and with exclusive
partners in more than
110 countries.

All contact data at seca.com