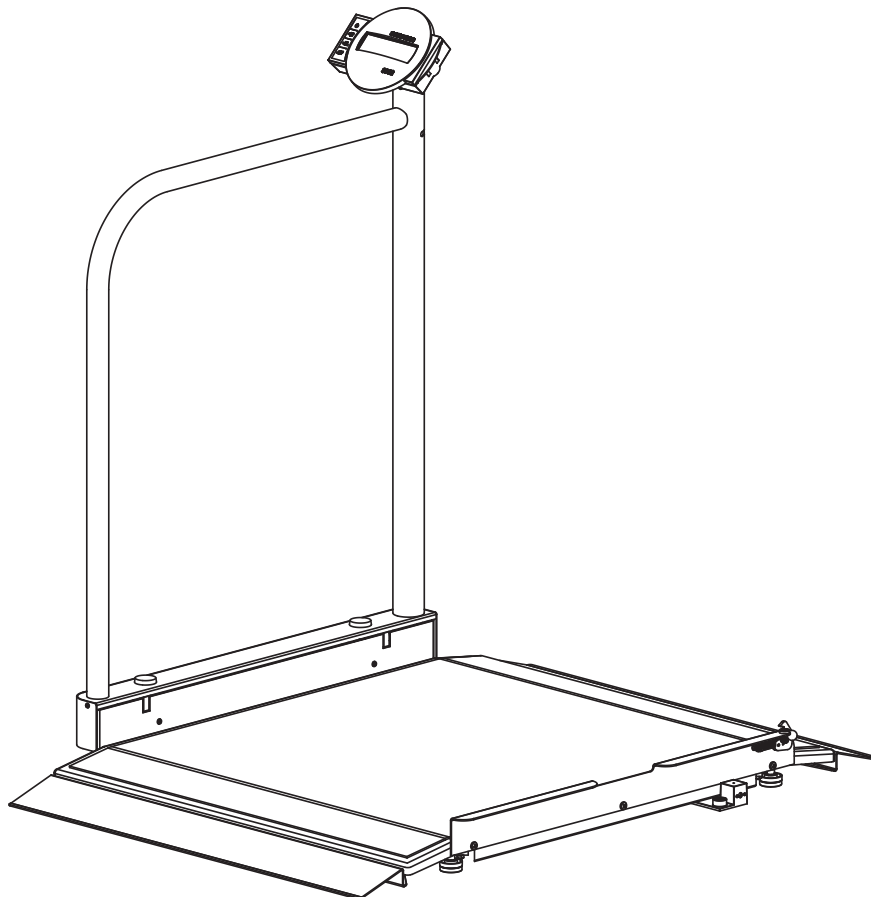
# seca 677/seca 676

## Instructions for use

17-10-07-681-100\_2025-08S

<b>Deutsch – Gebrauchsanweisung .....</b>	<b>3</b>
<b>English – Instructions for use .....</b>	<b>33</b>
<b>Français – Mode d’emploi.....</b>	<b>62</b>
<b>Italiano – Istruzioni per l’uso .....</b>	<b>92</b>
<b>Español – Instrucciones de uso .....</b>	<b>122</b>
<b>Português – Manual de instruções de utilização .....</b>	<b>152</b>
<b>Polski – Instrukcja obsługi.....</b>	<b>182</b>

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# TABLE OF CONTENTS

<b>1 About this document</b> .....	<b>34</b>		
1.1 Representation in text .....	34		
1.2 Representation in diagrams .....	34		
1.3 Devices in this document .....	34		
Devices in this document .....	34		
<b>2 Description of device</b> .....	<b>35</b>		
2.1 Intended use.....	35		
2.2 Contraindications .....	35		
2.3 Clinical benefit .....	35		
2.4 Patient target group.....	35		
2.5 User qualification.....	35		
Assembly .....	35		
Operation.....	35		
Administration/network operation .....	35		
2.6 Functional description .....	35		
Measuring weight .....	35		
External interface module (optional accessory) .....	35		
seca connect 103 software (optional accessory) .....	35		
<b>3 Safety information</b> .....	<b>36</b>		
3.1 Safety information in these instructions for use.....	36		
3.2 Basic safety information .....	36		
Handling device.....	36		
Preventing electric shock .....	37		
Preventing injuries and infections.....	37		
Preventing damage to device.....	38		
Handling measuring results .....	38		
Handling packaging material .....	39		
<b>4 Overview</b> .....	<b>39</b>		
4.1 Controls .....	39		
4.2 Symbols in the display .....	40		
4.3 Markings on the device and on the type plate.....	41		
4.4 Menu structure .....	43		
4.5 Navigating in the menu.....	43		
<b>5 Starting up device</b> .....	<b>44</b>		
5.1 Scope of delivery.....	44		
5.2 Fitting the display housing .....	45		
5.3 Fitting the ramps .....	45		
5.4 Establishing power supply .....	46		
5.5 Making the scale operational .....	46		
Setting up the handrail .....	46		
Aligning the scale .....	47		
5.6 Setting up the data connection (optional) ....	47		
5.7 Transporting the scale.....	48		
Lowering the handrail .....	48		
Moving the scale .....	48		
<b>6 Operation</b> .....	<b>49</b>		
6.1 Safety information .....	49		
6.2 Weighing.....	49		
Switching on the scale .....	49		
Weighing a patient.....	50		
Taring off additional weight (TARE) .....	50		
			Keep the measuring result on the display (HOLD) .....
			50
			Determining Body Mass Index .....
			51
			Switching the weighing range (verified models).....
			51
			Switching off the scale .....
			52
		6.3 Other functions (menu).....	52
		Deleting saved values automatically (ACLR).....	52
		Saving additional weight permanently (Pt) .....	52
		Activating Autohold function (AHOld).....	53
		Activating/deactivating beeps (bEEP) .....	53
		Setting filtering (FIL).....	53
		Non-verified models: Switching the weight unit (Unit) .....	54
		Restoring factory settings (rESet) .....	54
		<b>7 Hygiene treatment</b> .....	<b>55</b>
		7.1 Cleaning .....	55
		7.2 Disinfecting.....	55
		7.3 Sterilizing .....	55
		<b>8 Function check</b> .....	<b>55</b>
		<b>9 Troubleshooting</b> .....	<b>56</b>
		<b>10 Servicing</b> .....	<b>57</b>
		10.1 Verified scales .....	57
		10.2 Non-verified scales.....	58
		<b>11 Technical data</b> .....	<b>58</b>
		11.1 General technical data .....	58
		11.2 Weight measurement .....	59
		<b>12 Optional accessories and spare parts</b> .....	<b>60</b>
		<b>13 Disposing of the device</b> .....	<b>60</b>
		<b>14 Warranty</b> .....	<b>60</b>
		<b>15 Declaration of conformity</b> .....	<b>61</b>

# 1 ABOUT THIS DOCUMENT

## 1.1 Representation in text

Symbol	Description
✓	Requirement for actions
▶	Action
1. 2.	Actions with specified sequence
a) b)	Steps of an action with specified sequence
⇒	Result of an action
• •	First level of a list
– –	Second level of a list

## 1.2 Representation in diagrams

Symbol	Description
	Indicates directions of movement of the device or of device components
	Navigation path in menu displays
	<ul style="list-style-type: none"><li>• Correct action</li><li>• Correct result of action</li></ul>
	<ul style="list-style-type: none"><li>• Incorrect action</li><li>• Incorrect result of action</li></ul>
	End of a procedure, e.g. the installation of a part

## 1.3 Devices in this document

This document applies to the following devices:

- **seca 677**: Verified scale
- **seca 676**: Non-verified scale

## 2 DESCRIPTION OF DEVICE

### 2.1 Intended use

The multifunctional scale supports physicians in decision-making regarding diagnosis or therapy, based on weight or weight changes.

### 2.2 Contraindications

No contraindications are known.

### 2.3 Clinical benefit

The multifunctional scale supports physicians in decision-making regarding diagnosis or therapy, based on the measured weight.

By that, the multifunctional scale only provides an indirect clinical benefit.

The physician must perform additional examinations for an accurate diagnosis.

### 2.4 Patient target group

The scale is intended for persons of any age (with the exception of babies) who

- do not exceed the maximum weight capacity of the scale and
- use a wheelchair or
- are able to stand independently on the scale with the support of a handrail.

### 2.5 User qualification

**Assembly** Devices shipped partly assembled may only be assembled by sufficiently qualified persons (such as specialist dealers, hospital technicians or seca Service technicians).

**Operation** The device may only be operated by persons with formal training in the healthcare sector or in medicine.

**Administration/network operation** The device may only be set up and incorporated in a network by experienced administrators or hospital technicians.

### 2.6 Functional description

**Measuring weight** Weight calculation is performed by four load cells.

**External interface module (optional accessory)** Data can be sent to the **seca connect 103** software via a WiFi or LAN connection with the aid of the **seca 452** external interface module (optional).

**seca connect 103 software (optional accessory)** The **seca connect 103** software obtains measurement data from an internal or external interface module and sends them to compatible seca measuring devices or to third-party EMR systems.

## 3 SAFETY INFORMATION

### 3.1 Safety information in these instructions for use

---

**DANGER!**

Used to identify an extremely hazardous situation. If you fail to take note of this information, serious irreversible or fatal injuries will occur.

**WARNING!**

Used to identify an extremely hazardous situation. If you fail to take note of this information, serious irreversible or fatal injuries may result.

**CAUTION!**

Used to identify a hazardous situation. If you fail to take note of this information, minor to moderate injuries may result.

**NOTICE!**

Used to identify possible incorrect usage of the device. If you fail to take note of this information, you may damage the device, or the measuring results may be incorrect.

**NOTE**

Includes additional information about use of the device.

### 3.2 Basic safety information

---

**Handling device**

- ▶ Please take note of the information in these instructions for use.
- ▶ Keep the instructions for use in a safe place. The instructions for use are a component of the device and must be available at all times.
- ▶ In the interest of patient safety, you and your patients are obliged to report serious events that occur in connection with this product to the manufacturer and to the authority responsible in your country.

**DANGER!****Risk of explosion**

- ▶ Do not use the device in an environment in which one of the following gases has accumulated:
  - oxygen
  - flammable anesthetics
  - other flammable substances/air mixtures

**CAUTION!****Patient hazard, damage to device**

- ▶ Additional devices connected to medical electrical devices must provide evidence of compliance with the relevant IEC or ISO standards (e.g. IEC 60950 for data-processing devices). Furthermore, all configurations must comply with the requirements of standards for medical systems (see IEC 60601-1-1 or Section 16 of edition 3.1 of IEC 60601-1 respectively). Anyone connecting additional devices to medical electrical devices is considered a system configurator and therefore responsible for ensuring that the system complies with the requirements of standards for systems. This also applies to additional devices recommended by seca. Your attention is drawn to the fact that local laws take precedence over the above-mentioned requirements of standards. In the event of any queries, please contact your local specialist dealer or Technical Service.

 **CAUTION!**  
**Patient hazard, damage to device**

- ▶ Have servicing carried out regularly as described in the relevant section of this document.
- ▶ Technical modifications may not be made to the device. The device does not contain any user-serviceable parts. Only have servicing and repairs performed by an authorized seca service partner. You can find a service partner in your vicinity at [www.seca.com](http://www.seca.com) or by emailing [service@seca.com](mailto:service@seca.com).
- ▶ Use only seca original accessories and spare parts, otherwise seca will not grant any warranty.

 **CAUTION!**  
**Patient hazard, malfunction**

- ▶ Keep medical electrical devices such as high-frequency surgical equipment at a minimum distance of approx. 1 meter to prevent faulty measurements.
- ▶ Keep high-frequency equipment such as cellphones at a minimum distance of approx. 1 meter to prevent faulty measurements.
- ▶ The actual transmission output of HF devices may require minimum distances of more than 1 meter. For details, go to [www.seca.com](http://www.seca.com).

## Preventing electric shock

 **WARNING!**  
**Electric shock**

- ▶ Set up devices which can be operated with a power supply unit so that the power supply socket is within easy reach and the power supply can be disconnected quickly.
- ▶ Ensure that your local electricity supply matches the details on the power supply unit.
- ▶ Never touch the power supply unit with wet hands.
- ▶ Do not use extension cables and multiple outlets.
- ▶ Ensure that cables are not crushed or damaged by sharp edges.
- ▶ Ensure that cables do not come into contact with hot objects.
- ▶ Do not operate the device at an altitude of more than 3000 m above sea level.

## Preventing injuries and infections

 **WARNING!**  
**Injury from falling**

- ▶ Ensure that the device is steady and level.
- ▶ Route connector cables (if present) so that neither users nor the patient can trip over them.
- ▶ The device is not designed for supporting patients when getting up, e.g. from a wheelchair. Assist people with limited motor skills when they are getting up, e.g. from a wheelchair.
- ▶ Ensure that the patient does not step directly onto or off the edges of the weighing platform.
- ▶ Ensure that the patient steps onto and off the weighing platform slowly and safely.

 **WARNING!**  
**Danger of slipping**

- ▶ Ensure that the weighing platform is dry before the patient steps onto it.
- ▶ Ensure that the patient has dry feet before stepping onto the weighing platform.
- ▶ Ensure that the patient steps onto and off the weighing platform slowly and safely.

## Preventing damage to device

### **WARNING!** **Risk of infection**

- ▶ Before and after every measurement, wash your hands to reduce the risk of cross-contamination and nosocomial infections.
- ▶ Should the patient have contagious diseases, subject the device to a hygiene treatment as described in the relevant section of these instructions for use immediately afterwards.
- ▶ Ensure that the patient does not have any open wounds or infectious skin alterations which may come into contact with the device.
- ▶ Subject the device to a hygiene treatment at regular intervals.

### **NOTICE!** **Damage to device**

- ▶ Ensure that fluids never get inside the device. These can destroy the electronics.
- ▶ Switch off the device before you disconnect the power supply unit from the power supply socket.
- ▶ If the device is not to be used for an extended period, disconnect the power supply unit from the power supply socket. Only then is the device de-energized.
- ▶ Do not drop the device.
- ▶ Do not subject the device to shocks or vibrations.
- ▶ Perform a function check before each use as described in the corresponding section in this document. Do not operate the device if it is not working properly or is damaged.
- ▶ Do not place the device in direct sunlight and ensure that it is not placed in the direct proximity of a heat source. The excessive temperatures could damage the electronics.
- ▶ Avoid rapid temperature fluctuations. If the device is transported so that a temperature difference of over 20 °C occurs, the device must be left to stand for at least 2 hours before it is switched on, otherwise condensation may form; this may damage the electronics.
- ▶ Use the device only in the intended ambient conditions.
- ▶ Do not use the device over 3000 m above mean sea level. Electronic components might become damaged.
- ▶ Store the device only in the intended storage conditions.
- ▶ Use only cleaning agents and disinfectants that match the details in the section entitled "Hygiene treatment".
- ▶ Ensure that maximum capacity is not exceeded.

## Handling measuring results

### **CAUTION!** **Patient hazard**

To prevent misinterpretations, measuring results for medical purposes must only be displayed and used in SI units (kilograms/grams, meters/centimeters). Some devices have the option of displaying measuring results in different units. This is purely an additional function.

- ▶ Only use measuring results in SI units.
- ▶ The user takes sole responsibility for the use of measuring results in non-SI units.

**NOTICE!****Inconsistent measuring results**

- ▶ Before you save and continue using measured values determined using this device (e.g. in seca software or in an EMR system), ensure that the measured values are plausible.
- ▶ If measured values have been sent to seca software or to an EMR system, ensure before continuing to use them that the measured values are plausible and assigned to the correct patient.

**Handling packaging material****WARNING!****Risk of suffocation**

Packaging materials made of plastic film (bags) present a risk of suffocation.

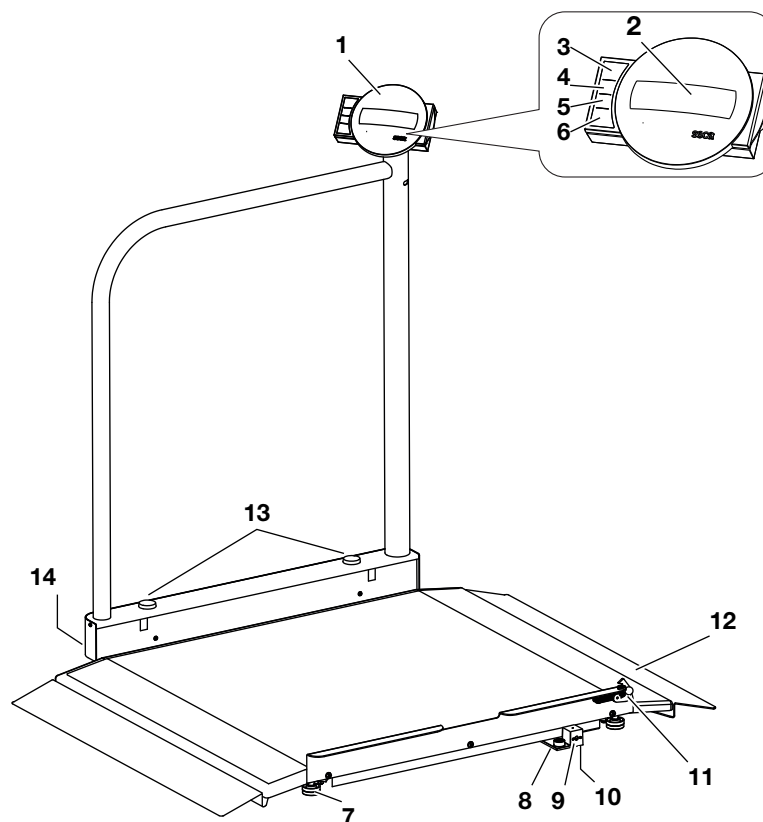
- ▶ Store packaging material out of the reach of children.
- ▶ If the original packaging material is no longer available, only use plastic bags with safety holes to reduce the risk of suffocation. Use recyclable materials if possible.

**NOTE**

Store the original packaging material for future use (e.g. returning for servicing).



## 4 OVERVIEW

### 4.1 Controls

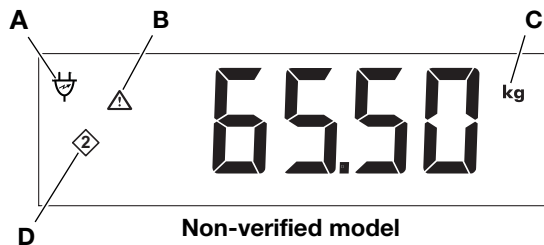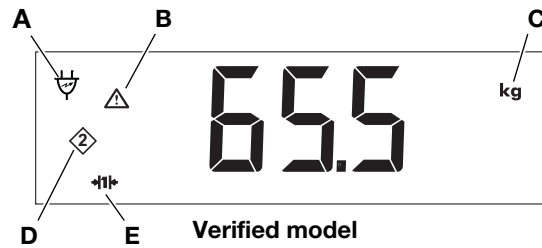Item	Control	Function
1	Display housing	Central control and display element
2	Display	Display element for measuring results and to configure the device
3		Switches the scale on and off
4		During weighing: <ul style="list-style-type: none"> <li>• Press briefly: Activate <b>Hold</b> function</li> <li>• Press and hold: Activate <b>Tare</b> function</li> </ul> In the menu: <ul style="list-style-type: none"> <li>• Select submenu, select menu item</li> </ul>
5		During weighing: <ul style="list-style-type: none"> <li>• Press briefly: Activate <b>BMI</b> function</li> <li>• Press and hold: Call up menu</li> </ul> In the menu: <ul style="list-style-type: none"> <li>• Select submenu, select menu item</li> </ul>
6		In the menu: <ul style="list-style-type: none"> <li>• Confirm selected menu item</li> <li>• Save set value</li> </ul>
7	Foot screw	4 pcs, for precise alignment
8	Spirit level	Indicates whether the device is horizontal
9	Power supply connection	Serves to connect the optional power supply unit
10	Display connection	Serves to connect the display head
11	Ramp	2 pcs, to access the scale e.g. with a wheelchair
12	Safety hook	To secure the handrail during transport
13	Locking screw	To secure the handrail during weighing
14	Caster	2 pcs, for transporting over short distances

## 4.2 Symbols in the display

### NOTE

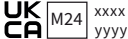
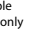
In the following chapters of these instructions for use, the display of the verified model is shown, unless a distinction between “verified/non-verified” is required.

Item	Control	Function
A		Operation with power supply unit
B		Non-verifiable function active (for technical reasons, symbol is also displayed on non-verified scales)
C	<b>kg</b> <b>lbs</b>	Unit in which weight is displayed ( <b>lbs</b> only on non-verified models)
D		Memory currently in use
E	→ 1 ← → 2 ←	Weighing range currently in use (only verified models)

### 4.3 Markings on the device and on the type plate

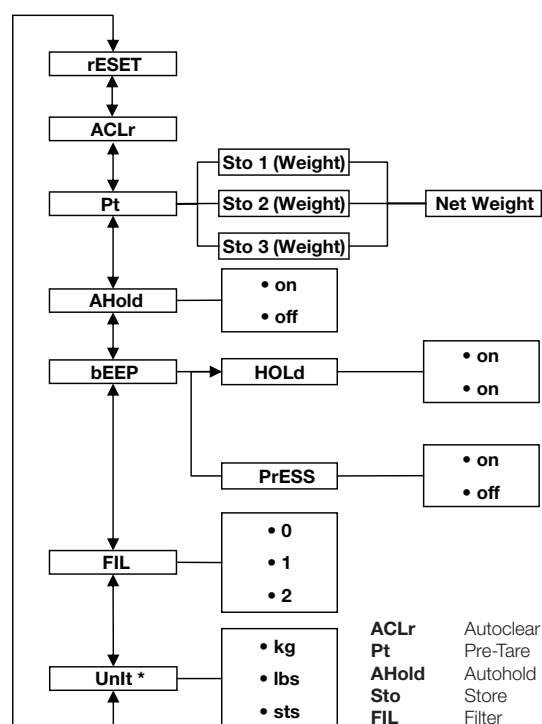
Markings on the device and on the type plate	
Symbol	Meaning
	Name and address of manufacturer, date of manufacture
UDI	Unique Device Identifier (product identification number)
	Article number
	Serial number
<b>GAL</b>	Value in $m/s^2$ (model-dependent) <ul style="list-style-type: none"> <li>• Gravitational acceleration on earth</li> <li>• Depends on the intended location</li> </ul>
<b>ProdID</b>	Product identification number
<b>Mat.No.</b>	Variant number
<b>Approval Type</b>	Type designation of design approval
	Follow instructions for use
	Medical electrical device, Type B
<b>IP20</b>	Type of protection to IEC 60529: <ul style="list-style-type: none"> <li>• Protection against ingress of solid foreign bodies with a diameter of over 12.5 mm</li> <li>• Protection against access with fingers</li> <li>• No protection from ingress of water</li> </ul>
<b>e</b>	Value in units of mass (verified models) Used to classify and verify a scale
<b>d</b>	Value in units of mass (non-verified models) States the difference between two consecutive display values
→ x ←	Active weighing range
	Class III scale to directive 2014/31/EU

Markings on the device and on the type plate	
Symbol	Meaning
	<p>Device complies with EU directives</p> <ul style="list-style-type: none"> <li>• <b>M</b>: Conformity label in compliance with directive 2014/31/EU for non-automatic scales (verified models)</li> <li>• <b>24</b>: (Example: 2024) Year in which declaration of conformity was implemented and the CE-label was applied (verified models)</li> <li>• <b>0102</b>: Notified Body for Metrology (verified models)</li> <li>• <b>0123</b>: Notified Body for Medical Devices</li> </ul>
	Medical device in accordance with regulation (EU) 2017/745
	Device meets the requirements of the USA and Canada. Certified and tested by a licensing laboratory (NRTL) of TÜV SÜD Product Services GmbH.
	<p>Device complies with United Kingdom directives</p> <ul style="list-style-type: none"> <li>• <b>M</b>: Conformity label in compliance with UK directive SI 2016 no. 1152 for non-automatic scales (NAWIR) (verified models)</li> <li>• <b>24</b>: (Example: 2024) Year in which declaration of conformity was implemented and the UKCA label was applied (verified models)</li> <li>• <b>xxxx</b>: Notified Body for Medical Devices of the United Kingdom</li> <li>• <b>yyyy</b>: Notified Body for Metrology of the United Kingdom (verified models)</li> </ul>
	<p>Importer/representative in the United Kingdom:</p> <p>seca Ltd 40 Barn Street B5 5QB Birmingham United Kingdom</p>
	<p>Importer/representative in Switzerland:</p> <p>seca ag (schweiz) Medizinische Waagen und Messsysteme Schönmatte Str. 2 CH-4153 REINACH</p>
	<p>Type plate on the power supply connection socket</p> <ul style="list-style-type: none"> <li>• Power supply voltage required in V</li> <li>• Maximum current consumption in mA</li> <li>• : Note polarity of device connector</li> <li>• : Operate device with direct current</li> <li>• : Use compatible seca power supply units only</li> </ul>
	Do not dispose of device in household waste

Markings on the packaging	
Symbol	Meaning
	Protect from moisture
	Arrows indicate top of product Transport and store in an upright position
	Fragile, do not throw or drop

Markings on the packaging	
Symbol	Meaning
	Permitted min. and max. temperature for transport and storage
	Permitted min. and max. humidity for transport and storage
	Permitted min. and max. air pressure for transport and storage
	Open packaging here
	Packaging material can be disposed of through recycling programs

## 4.4 Menu structure

\* Unverified scales

## 4.5 Navigating in the menu

AHOLD

FIL

1. Switch on the scale.
2. Hold down the arrow key ▼ until the menu is called up.  
⇒ The last menu item selected is displayed (in this case: Autohold **AHOLD**).
3. Press one of the arrow keys until the desired menu item appears in the display (in this case: filtering **FIL**).

4. Confirm the selection by pressing the  $\blacktriangleleft$  key.  
 ⇒ The current setting for the menu item or a submenu is displayed (in this case: level **0**).

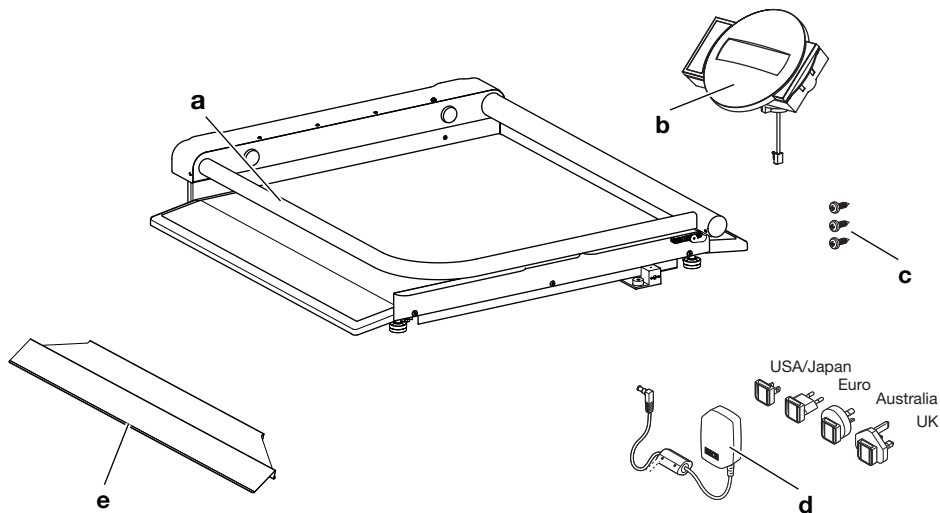
5. To change the setting, press the  $\blacktriangle$  or the  $\blacktriangledown$  key until the desired setting is displayed (in this case: level **2**).
6. Confirm the selection by pressing the  $\blacktriangleleft$  key.  
 ⇒ The menu closes.
7. To make more settings, call up the menu again and proceed as described.

**NOTE**

If no key is pressed for approx. 24 seconds, the menu will close.


## 5 STARTING UP DEVICE

### 5.1 Scope of delivery


Item	Component	Pcs.
<b>a</b>	Weighing platform with handrail folded down	1
<b>b</b>	Display housing with connecting cable	1
<b>c</b>	Screw, 3.5 x 9.5 mm	3
<b>d</b>	Power supply unit with adapters (depending on model: Power supply unit with Euro connector)	1
<b>e</b>	Ramp	2
–	Phillips screwdriver, size 1, not shown	1
–	Instructions for use, not shown	1

## 5.2 Fitting the display housing

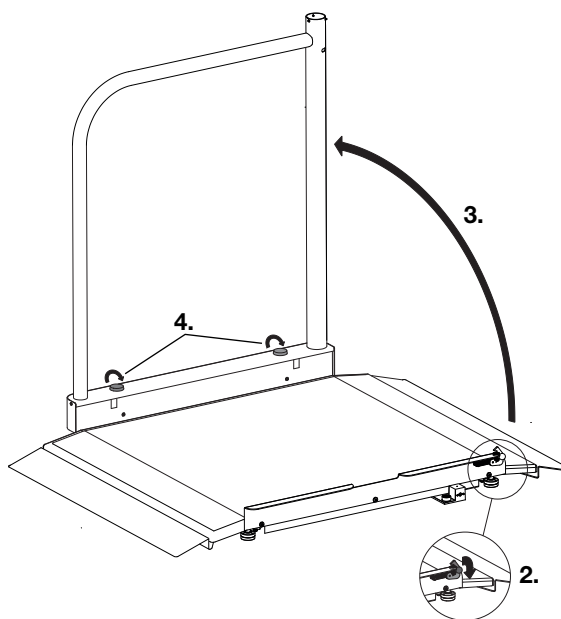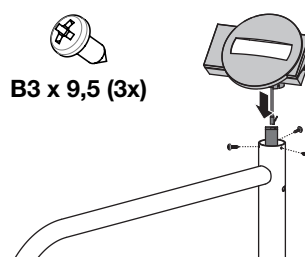1. Place the scale on a solid, level surface.
2. Release the latch for the handrail.
3. Set up the handrail and hold it firmly.
4. Tighten up both locking screws until the handrail is firm without any play.

5. Take the display cable out of the handrail fixing
6. Connect the display cable in the handrail to the cable of the display housing.
7. Push the cable carefully into the handrail.
8. Place the display housing on the handrail.
9. Ensure that the bores on the display housing and the bores in the handrail are in line over one another.
10. Screw the display housing to the handrail (1 Nm).

## 5.3 Fitting the ramps

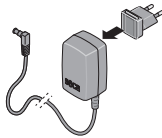
- Fit the supplied ramps as described in the relevant assembly instructions.

## 5.4 Establishing power supply


**⚠ WARNING!**  
**Personal injury or damage to the device as a result of incorrect power supply units**

Conventional power supply units may deliver a higher voltage than is indicated on them. The measuring device may overheat, catch fire, melt or short-circuit.

► Use only original seca power supply units with a controlled 12 V output voltage.

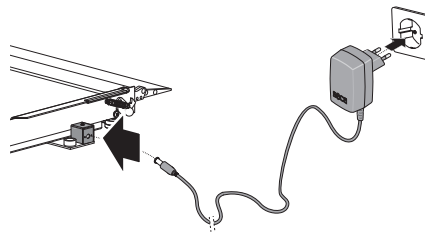
1. If required, insert the power supply connector necessary for your power supply into the power supply unit.

2. Insert the device connector of the power supply unit into the connecting socket of the scale.
3. Plug the power supply unit into a power supply socket.

## 5.5 Making the scale operational
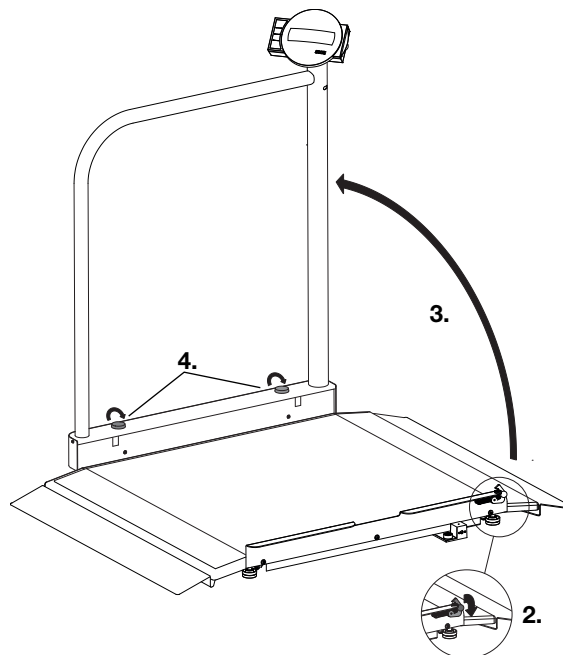
### Setting up the handrail

1. Place the scale on a solid, level surface.
2. Release the latch for the handrail.
3. Set up the handrail and hold it firmly.
4. Tighten up both locking screws until the handrail is firm without any play.
5. Insert the device connector of the power supply unit into the connection socket of the scale.

- Plug the power supply unit into a power supply socket.

### Aligning the scale

#### CAUTION!

##### Faulty measurement as a result of devices being set up incorrectly

Soft floors (wooden boards, for example) give under the patient's weight and falsify the measuring result. Incorrect ambient conditions or lack of alignment lead to measurement errors.

- ▶ Select a setup location with a level, stable floor in order to achieve precise measuring results.
- ▶ Use the device only in the intended ambient conditions outlined in the section entitled "Technical data".
- ▶ Align the device before using it and every time it changes location.

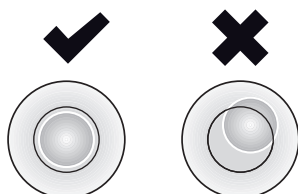
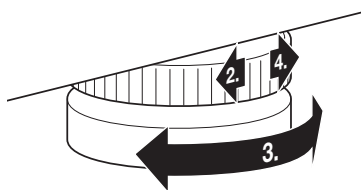
#### CAUTION!

##### Faulty measurement as a result of devices being set up incorrectly

If the weighing platform or attachments (e.g. ramps, handrails, rail parts) come into contact with walls or objects, the measuring result will be falsified.

- ▶ Locate the device at a sufficient distance from walls and objects.

- Place the scale on a firm, level surface.
- Undo the knurled wheels.


- Align the device by turning the foot screws.  
The air bubble in the spirit level must be located in the exact center of the circle.
- Tighten the knurled wheels in the direction of the arrow.  
⇒ The foot screws are secured against being adjusted.

## 5.6 Setting up the data connection (optional)

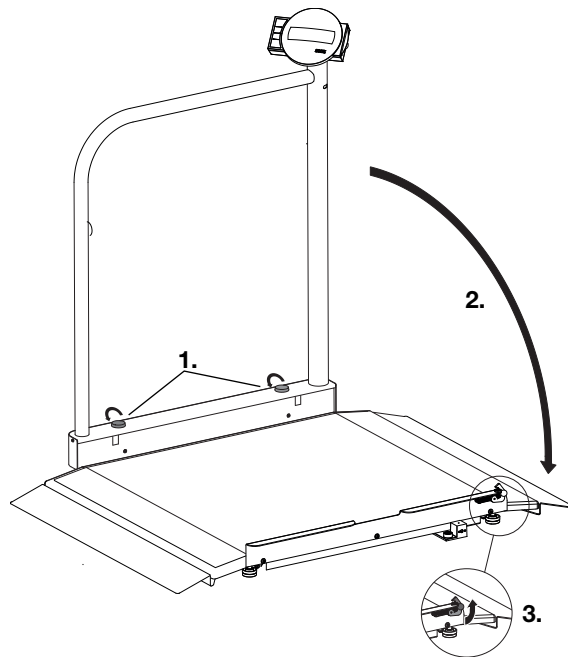If you want to set up a data connection via **seca connect 103** and **seca 452**, contact your hospital technician, your IT administrator or seca Service.

## 5.7 Transporting the scale

### Lowering the handrail


1. Undo the locking screws.
2. Carefully fold down the handrail until it rests on the weighing platform.
3. Fold up the latch until it engages securely in the handrail.

### Moving the scale
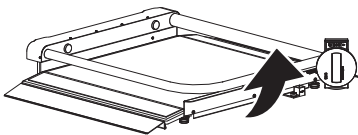
#### **WARNING!** Injury from device falling over

- ▶ Always transport the scale with two persons.
- ▶ Transport the scale carefully to prevent it from tipping over or colliding with other people.
- ▶ Keep the scale in a safe place and prevent it from tipping over. Do not leave the scale unsecured in an upright position.

1. Disconnect the power supply unit from the power supply socket.
2. Disconnect the device connector of the power supply unit from the connection socket of the scale.
3. Tilt the device until it can be moved freely on the casters.
4. Move the device in this position to the desired set up or storage location.

## 6 OPERATION

### 6.1 Safety information

---

**WARNING!**

**Injury from falling**

- ▶ Keep an eye on ramps, edges and the guardrail when pushing the handling aid onto the weighing platform.
- ▶ Make sure that the brakes on patient handling aids are engaged during the weighing procedure.

**WARNING!**

**Injury from falling**

Patients in motorized wheelchairs might drive onto the weighing platform too quickly or at the wrong angle and fall.

- ▶ Help patients in motorized wheelchairs onto the weighing platform. These patients should not drive onto the weighing platform unassisted.

**WARNING!**

**Injury from falling/damage to device**

- ▶ If the patient handling aid has a quick-locking mechanism or similar parts close to the floor, ensure that such parts do not get caught on the scale or scratch the weighing platform.

**WARNING!**

**Faulty measurement (patient handling aid)**

- ▶ Make sure that all wheels of the patient handling aid are standing on the surface sensitive to weighing. Make sure that the patient handling aid does not touch the wall or any other objects.
- ▶ Ensure that the wheels of patient handling aids are not positioned at the transitions between the weighing platform and ramps (additionally with the bed scale: between the two weighing platforms), so producing a force shunt (incorrect measured values).

**CAUTION!**

**Faulty measurement**

- ▶ Help patients in (motorized) wheelchairs onto the scale in order to position the wheelchair correctly. Patients should not perform measurement themselves.

**CAUTION!**

**Personal injury/faulty measurement**

- ▶ Before using the device each time, perform a function check as described in the corresponding section of these instructions for use.

### 6.2 Weighing

---

#### Switching on the scale

- ▶ Press the  key.
  - ⇒ All the elements of the display are shown briefly, and then **SECA** appears in the display.
  - ⇒ The scale is operational when **0.0** (non-verified model: **0.00**) appears in the display.

## Weighing a patient

### CAUTION! Injury from falling

Persons with limited mobility may fall when stepping on or off the weighing platform.

- ▶ Support people with limited mobility when they step on and off the scale.
- ▶ Move people who cannot stand without assistance onto the weighing platform using a suitable patient handling aid.

✓ The weighing platform has no load.

1. Start the weighing procedure:

- ▶ Move the patient onto the weighing platform with a suitable patient handling aid
- ▶ Ask the patient to step onto the weighing platform

2. Ask the patient not to move.

3. Read off the measuring result.

### NOTE

To weigh patients with limited mobility, refer to section → [Taring off additional weight \(TARE\), page 50](#). To weigh patients who are not mobile, refer to section → [Saving additional weight permanently \(Pt\), page 52](#).

## Taring off additional weight (TARE)

Use the **TARE** function to prevent an additional weight affecting the patient's weight value.

This function is suitable for patients with limited mobility who cannot stand for the entire weighing procedure. You can start by weighing a wheelchair, for example, and taring off its unladen weight. The patient can then be weighed in a sitting position.

### NOTICE!

#### Faulty measurement as a result of force shunt

If an additional weight is touching the surface on which the scale is standing, the weight will not be measured correctly.

- ▶ Make sure that additional weights are only placed on the scale's weighing platform.

1. Switch on the scale.

2. Place the wheelchair on the scale.

3. Hold down the ▲ **hold/tare** arrow key until the **NET** message appears in the display.

4. Wait until the display stops flashing and **0.0** appears instead.

⇒ The weight of the wheelchair is tared off.

5. Guide the patient onto the weighing platform and support them to sit in the wheelchair.

6. Ask the patient not to move.

⇒ The patient's weight is displayed.

7. Read off the measuring result.

8. To deactivate the **TARE** function, press the **hold/tare** arrow key until the **NET▲** message is no longer displayed or turn off the scale.

### NOTE

The maximum weight which can be displayed is reduced by the weight of the objects which have been tared.

## Keep the measuring result on the display (HOLD)

When the **Hold** function is activated, the weight is still displayed after the load has been removed from the scale. This enables you to attend to the patient before recording the weight value.

✓ The scale has no load.

1. Switch on the scale.

2. Weigh the patient as described in the section entitled → [Weighing a patient, page 50](#).

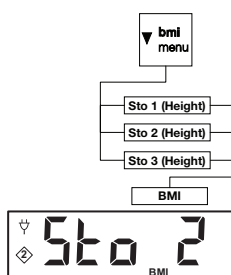
3. Press the **▲ hold/tare** arrow key briefly.
4. Wait until the weight value has stopped flashing.
  - ⇒ The **▲** symbol (non-verifiable function) and the **HOLD** message are displayed.
  - ⇒ The patient can leave the weighing platform and the weight value will continue to be displayed.
5. To deactivate the function, press the **▲ hold/tare** arrow key briefly.
  - ⇒ The **▲** symbol and the **HOLD** message are no longer displayed.
  - ⇒ The weight value is no longer displayed.

**NOTE**

If the autohold function is activated, the weight value is automatically displayed permanently until the scale switches itself off or is switched off, see → [Activating Autohold function \(AHOld\)](#), page 53.

**Determining Body Mass Index**

Body Mass Index correlates height and weight. To do this, the patient's height must be entered manually. The device calculates the BMI from the entered value and the measured weight.

You can save heights for a maximum of three patients so that the values are quickly available for repeat measurements.

✓ The scale has no load.

1. Switch on the scale.
2. Press the **▼ bmi/menu** arrow key briefly.
  - ⇒ The message **BMI** appears.
  - ⇒ The last memory used is displayed (in this case, memory 2).
3. You have the following options for continuing:
  - ▶ Adopt memory
  - ▶ Select different memory: Press the **▲** or the **▼** key
4. Confirm your setting by pressing the **↵** key.
  - ⇒ Arrows flash in the display.
  - ⇒ The last height saved to the selected memory is displayed.
5. You have the following options for continuing:
  - ▶ Adopt value
  - ▶ Change value: Press the **▲** or the **▼** key
6. Confirm your setting by pressing the **↵** key.
  - ⇒ The height entered is saved and can be called up again for calculating a new BMI.

**NOTE**

Make a note of the memory and assigned height so that you can call up the value again for calculating a new BMI.

7. Weigh the patient as described in the section entitled "Weighing a patient".
  - ⇒ The patient's BMI is automatically calculated and displayed.
8. To deactivate the BMI function, briefly press the **↵** key.

**Switching the weighing range (verified models)**

The scale has two weighing ranges. In weighing range 1 (→|1|←), graduations in the weight display are finer. In weighing range 2 (→|2|←), you can use the maximum capacity of the scale.

After the scale is switched on, weighing range 1 is active.

If a particular weight value is exceeded, the scale automatically switches to weighing range 2.

- ▶ To switch back to weighing range 1, completely remove the weight from the scale.
  - ⇒ Weighing range 1 is active again.

Switching off the scale ► Press the  key.

## 6.3 Other functions (menu)

### Deleting saved values automatically (ACLr)

Out-of-date measuring results lead to incorrect calculation of BMI. You can set the device so that the measuring results are deleted automatically after 5 minutes.

#### NOTE

This function is activated at the factory on some models. The function can be deactivated.

1. From the menu, select the **ACLr** menu item.
2. Confirm your selection.  
⇒ The current setting is displayed.
3. Select the desired setting:
  - **On**
  - **OFF**
4. Confirm your selection.  
⇒ The menu closes.

### Saving additional weight permanently (Pt)

The **Pre-Tare** function is suitable for patients who are not mobile and need to be weighed in a wheelchair, for example. You can save the unladen weight of the wheelchair independently of a weighing procedure. During the weighing procedure, you can call up the unladen weight of the wheelchair and have it automatically deducted from the measuring result.

The device has three memories for weight values. You can save various weight values and call them up individually depending on the initial situation, so that they are automatically deducted from the measuring result.

1. From the menu, select the **Pt** menu item.
2. Confirm your selection.  
⇒ The last memory used is displayed.
3. You have the following options for continuing:
  - Adopt memory
  - Select different memory: Press the **▲** or the **▼** key
4. Confirm your selection.

- ⇒ Arrows flash in the display.  
⇒ The additional weight saved to the selected memory is displayed.
5. You have the following options for continuing:
  - Adopt value
  - Change value: Press the **▲** or the **▼** key

#### NOTE

When you enter the value **0**, the function is switched off. The message **Pt** is no longer shown in the display.

6. Confirm your selection.
7. Move the patient onto the scale.
  - ⇒ The patient's weight is displayed.
  - ⇒ The saved additional weight has been deducted automatically.
8. To deactivate the function, select the **Pt** menu item from the menu again.
9. Confirm your selection.
  - ⇒ The function is deactivated.
  - ⇒ The menu closes.

**NOTE**

When you switch the scale off, the function is switched off. The message **Pt** is no longer displayed when you switch the scale on again.

**Activating Autohold function (AHOLD)**

When you activate the **Autohold** function, the measuring result is displayed for every weighing procedure after the weight has been removed from the scale. It is then no longer necessary to activate the **Hold** function manually.

**NOTE**

This function is activated at the factory on some models. The function can be deactivated.

AHOLD

1. From the menu, select the **AHOLD** menu item.
2. Confirm your selection.  
⇒ The current setting is displayed.
3. Select the desired setting:
  - **On**
  - **OFF**
4. Confirm your selection.  
⇒ The setting is adopted.  
⇒ The menu closes.

**Activating/deactivating beeps (bEEP)**

You can activate beeps for a number of device functions:

bEEP

PrESS

1. From the menu, select the **bEEP** menu item.
2. Confirm your selection.
3. Select a menu item:
  - **PrESS**: beep when key is pressed
  - **HOLD**: beep when weighing value is stable
4. Confirm your selection.  
⇒ The current setting is displayed.
5. Select the desired setting:
  - **On**
  - **OFF**
6. Confirm your selection.  
⇒ The setting is adopted.  
⇒ The menu closes.

**Setting filtering (FIL)**

Setting the filter (**FIL** = filter) allows you to reduce interference when determining weight. The selected setting affects the sensitivity with which the weight display reacts to patient movements and the time until the **Hold** function displays a weight value permanently (**Hold** value).

Setting (FIL)	Sensitivity	Determination of Hold value
0	Sensitive	Slow
1	Medium	Medium
2	Insensitive	Fast

**NOTE**

With the **0** setting and patients who are not very steady on their feet, it is possible that no weight value is displayed permanently despite the **Hold** function being activated.

The greatest deviation between weight value displayed and actual value occurs at setting **2**.

FIL

1. From the menu, select the **FIL** menu item.
2. Confirm your selection.  
⇒ The current setting is displayed.

3. Select a filtering level.
4. Confirm your selection.
  - ⇒ The setting is adopted.
  - ⇒ The menu closes.

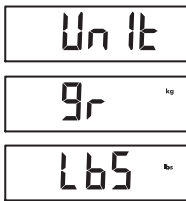
**Non-verified models:  
Switching the weight unit  
(UnIt)**

You can select the unit in which weight is to be displayed.

**⚠ CAUTION!  
Patient hazard**

To prevent misinterpretations, measuring results for medical purposes must only be displayed and used in SI units (kilograms/grams, meters/centimeters). Some devices have the option of displaying measuring results in different units. This is purely an additional function.

- ▶ Only use the measuring results in SI units.
- ▶ You as the user take sole responsibility for the use of measuring results in non-SI units.

1. From the menu, select the **UnIt** menu item.
2. Confirm your selection.
  - ⇒ The current setting is displayed.
3. Select the unit in which weight is to be displayed:
  - Kilograms/grams (**gr**)
  - Pounds (**lbS**)
  - Stones (**sts**)
4. Confirm your selection.
  - ⇒ The setting is adopted.
  - ⇒ The menu closes.

**Restoring factory settings  
(rESEt)**

You can restore the factory settings for the functions below:

Function	Factory setting
Autohold ( <b>AHOLd</b> )	<b>model-depend-ent</b>
Acoustic signal ( <b>PrESS</b> )	<b>OFF</b>
Acoustic signal ( <b>HOLd</b> )	<b>On</b>
Filtering ( <b>FIL</b> )	<b>0</b>
Autoclear ( <b>ACLr</b> )	<b>model-depend-ent</b>
Pre-tare ( <b>Pt</b> )	<b>0 kg</b>
Height for Body Mass Index ( <b>bMi</b> )	<b>170 cm</b>
Unit, weight ( <b>UnIt</b> )	<b>kg (gr)</b>

1. From the menu, select the **rESEt** menu item.
2. Confirm your selection.
  - ⇒ The menu closes.
3. Switch off the scale.
  - ⇒ Factory settings are restored and are available when the scale is switched on.

## 7 HYGIENE TREATMENT

### **WARNING!** **Electric shock**

The device is not de-energized when the on/off button is pressed and the display goes out. Use of fluids on the device may cause an electric shock.

- ▶ Before each hygiene treatment, ensure that the device is switched off.
- ▶ Disconnect the power supply connector before each hygiene treatment.
- ▶ Ensure that no fluids penetrate the device.

### **WARNING!** **Risk of infection**

- ▶ Subject the device to a hygiene treatment at regular intervals as described in this section.

### **NOTICE!** **Damage to device**

Unsuitable cleaning agents and disinfectants may damage the sensitive surfaces of the device and impair its operability.

- ▶ Do not use aggressive or abrasive cleaning agents.
- ▶ Do not use organic solvents (e.g. white spirit or petroleum spirit).

### 7.1 Cleaning

- ▶ If required, moisten a soft cloth with a mild soap solution and wipe the device over with it.

### 7.2 Disinfecting

1. Disinfect the device at regular intervals with a disinfectant suitable for sensitive surfaces and acrylic glass (e.g. 70 % ethanol).
2. Follow the instructions for use of the disinfectant.
3. Disinfect the device:
  - ▶ Moisten a soft cloth with disinfectant and wipe down the device with it.
  - ▶ Comply with the intervals, see table:

Interval	Component
Before and after each measurement	Weighing platform
As required	Keyboard, display, handrail

### 7.3 Sterilizing

The device must not be sterilized.

## 8 FUNCTION CHECK

- ▶ Perform a function check before each use.

A complete function check includes:

- A visual inspection for mechanical damage

- A test of the alignment of the device
- A visual and function check of the display elements
- A function check of all the controls shown in the section entitled “Overview”
- A function check of the optional accessories

If you find faults or deviations during the function check, first try to remedy the fault with the aid of the “Troubleshooting” section in this document.

**CAUTION!**

**Personal injury**

If you find faults or deviations during the function check which you are unable to remedy with the aid of the “Troubleshooting” section in this document, you must not use the device.

- ▶ Have the device repaired by seca Service or an authorized service partner.
- ▶ Follow the section entitled “Servicing” in this document.

## 9 TROUBLESHOOTING

**NOTICE!**

**Malfunction**

If the device is equipped with the optional **seca 452** external interface module and is connected to the **seca connect 103** software, error messages and error symbols which are not described here may occur.

- ▶ Follow the system instructions for use for **seca 103/452**.

If faults occur when operating the device, first attempt to remedy them yourself using the following table. If the fault persists, contact seca Service.

Fault	Cause	Remedy
No weight is displayed with a load on the scale.	The device has no power supply.	<ul style="list-style-type: none"> <li>• Switch on device.</li> <li>• Connect the power supply unit.</li> </ul>
<b>0.0</b> (verified models)/ <b>0.00</b> (non-verified models) is not displayed before weighing takes place.	A load was already on the device before it was switched on.	<ul style="list-style-type: none"> <li>• Remove the load from the device.</li> <li>• Switch off the device, then switch it back on again.</li> </ul>
----- is displayed before weighing takes place.	A load was already on the device before it was switched on.	<ul style="list-style-type: none"> <li>• Remove the load from the device.</li> <li>• Switch off the device, then switch it back on again.</li> </ul>
The weight display flashes.	Consistent weight value not yet detected.	<ul style="list-style-type: none"> <li>• One of the following functions is active: <b>Hold, Autohold, Tare</b>. Wait until the device has detected a consistent weight value. The display stops flashing.</li> <li>• Take the load off the device, wait until <b>0.0/0.00</b> is displayed and measure again.</li> </ul>
Implausible measuring results.	Patient moved too much during the measurement.	Take the load off the device, wait until <b>0.0/0.00</b> is displayed and repeat the measurement procedure.
One display segment is displayed continuously or not at all.	The display is defective.	Inform seca Service.
<b>StOP</b> is displayed.	Maximum capacity has been exceeded.	Remove the load from the device.

Fault	Cause	Remedy
tEMP is displayed.	The ambient temperature of the scale is too high or too low.	<ul style="list-style-type: none"> <li>Operate the device only in the intended ambient conditions (General technical data).</li> <li>After transporting or storing the device, wait approx. 15 minutes before switching it on.</li> </ul>
The device no longer reacts when keys are pressed.	Device is in an undefined state following implausible input.	<ul style="list-style-type: none"> <li>Take power supply unit out of the power supply socket.</li> <li>Wait approx. 1 minute.</li> <li>Plug the power supply unit into the power supply socket; the scale and the multifunctional display will switch on automatically.</li> </ul>
	The keypad is defective.	Inform seca Service.

## 10 SERVICING

### 10.1 Verified scales

#### CAUTION!

**Faulty measurements as a result of verification being performed poorly or not at all**

- ▶ Have verification performed only by authorized persons.
- ▶ Always have verification performed if one or more quality seals are damaged.

seca recommends having your device serviced prior to verification.

#### CAUTION!

**Faulty measurements as a result of poor servicing**

- ▶ Have servicing and repairs carried out exclusively by seca Service or an authorized service partner.
- ▶ You can find a service partner in your vicinity at [www.seca.com](http://www.seca.com) or by emailing [service@seca.com](mailto:service@seca.com).

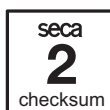
Have authorized persons perform verification in line with national legal regulations.

Verification is necessary whenever one or more quality seals are damaged or the contents of the verification counter no longer match the number on the applicable verification sticker. If quality seals are damaged, contact seca Service directly.

Verifications may only be performed by authorized agencies. To guarantee this, the scale is equipped with a verification counter which records each change in verification-related data.

If you want to check whether the scale has been properly verified, proceed as follows:

1. Ensure that the device is switched off.
2. Press and hold any key and start the scale.  
⇒ The current content of the verification counter flashes in the display for a few seconds.
3. Compare the content of the verification counter output with the number given on the verification counter seal.

Both numbers have to match for the verification to be valid. If the verification sticker and the verification counter do not match, the scale must be verified. Please contact your service partner or seca Service. Once the scale has been verified, a new, updated verifi-

cation counter sticker is used to identify the verification counter reading. The person authorized to perform verification secures this verification with an additional seal. The verification counter sticker can be ordered from seca Service.

## 10.2 Non-verified scales

The product needs to be set up carefully and serviced regularly. Depending on how frequently the product is used, seca recommends servicing at intervals of 3 to 5 years.

### CAUTION!

#### Faulty measurements as a result of poor servicing

- ▶ Have servicing and repairs carried out exclusively by seca Service or an authorized service partner.
- ▶ You can find your local service partner at [www.seca.com](http://www.seca.com).

## 11 TECHNICAL DATA

### 11.1 General technical data

Parameter	Value
Dimensions, total <ul style="list-style-type: none"> <li>• Depth</li> <li>• Width (incl. ramps)</li> <li>• Height (operation/transport)</li> </ul>	<p style="text-align: right;">919 mm 1176 mm 1120 mm/160 mm</p>
Dimensions, weighing platform <ul style="list-style-type: none"> <li>• Depth</li> <li>• Width</li> <li>• Height</li> </ul>	<p style="text-align: right;">800 mm 965 mm 57 mm</p>
Weight (incl. ramps)	approx. 41 kg
Height of digits, display	25 mm
Ambient conditions, operation <ul style="list-style-type: none"> <li>• Temperature</li> <li>• Air pressure</li> <li>• Humidity</li> </ul>	<p style="text-align: right;">+10 °C to +40 °C / +50 °F to 104 °F 700 – 1060 hPa 30 % – 80 % no condensation</p>
Ambient conditions, storage/transport <ul style="list-style-type: none"> <li>• Temperature</li> <li>• Air pressure</li> <li>• Humidity</li> </ul>	<p style="text-align: right;">-10 °C to +65 °C / +14 °F to 149 °F 700 – 1060 hPa 0 % – 95 % no condensation</p>
Medical device in accordance with regulation (EU) 2017/745	Class I with measuring function
IEC 60601-1: Medical electrical device, Type B	
Type of protection in accordance with IEC 60529	<b>IP20</b>
Duty cycle	Continuous duty
Service life, from commissioning <sup>a</sup>	at least 10 years
Application parts according to IEC 60601-1	Weighing platform, handrail, display housing, film keypad, display

Parameter	Value
Power supply unit	
<ul style="list-style-type: none"> <li>• Supply voltage</li> <li>• Current consumption</li> <li>• Insulated device, protection class II (IEC 60601-1)</li> <li>• Mains voltage</li> <li>• Power supply frequency</li> </ul>	<p>12 V approx. 20 mA</p> <p><input type="checkbox"/></p> <p>100 V – 240 V 50 Hz – 60 Hz</p>

<sup>a</sup> Minimum service life when used as intended and serviced in a timely manner. seca products usually have a significantly longer service life.

## 11.2 Weight measurement

Verified model	
Verification in line with directive 2014/31/EU	Class III
Maximum capacity	
<ul style="list-style-type: none"> <li>• Weighing range 1</li> <li>• Weighing range 2</li> </ul>	<p>200 kg 300 kg</p>
Minimum capacity	
<ul style="list-style-type: none"> <li>• Weighing range 1</li> <li>• Weighing range 2</li> </ul>	<p>2 kg 4 kg</p>
Graduation	
<ul style="list-style-type: none"> <li>• Weighing range 1</li> <li>• Weighing range 2</li> </ul>	<p>100 g 200 g</p>
Tare range	300 kg (subtractive)
Accuracy on initial verification	
<ul style="list-style-type: none"> <li>• Weighing range 1: &lt; 50 kg</li> <li>• Weighing range 1: 50 kg to 200 kg</li> <li>• Weighing range 2: &lt; 100 kg</li> <li>• Weighing range 2: 100 kg to 300 kg</li> </ul>	<p>± 50 g ± 100 g ± 100 g ± 200 g</p>

Non-verified model	
Maximum capacity	360 kg/800 lb/57 st
Minimum capacity	1 kg
Graduation	50 g/0.1 lb
Tare range	360 kg/800 lb/57 st (subtractive)
Accuracy	
<ul style="list-style-type: none"> <li>• &lt; 67 kg</li> <li>• 67 kg to 360 kg</li> <li>• &lt; 147.7 lb</li> <li>• 147.7 lb to 800 lb</li> <li>• &lt; 10.55 st</li> <li>• 10.55 st to 57 st</li> </ul>	<p>± 100 g ± 0.15 % ± 0.2 lb ± 0.15 % ± 0.2 lb ± 0.15 %</p>
At ambient temperature of 20° C and humidity of 45 %	

## 12 OPTIONAL ACCESSORIES AND SPARE PARTS

Accessory/spare part	Article number
Power supply unit with Euro connector: 230 V~ /50 Hz/ 12 V= / 130 mA	68 32 10 252
Switch-mode power supply unit with adapters: • Input: 100 V – 240 V~, 50 Hz – 60 Hz • Output: 12 V= / 0.98 A	68 32 10 265 009
Switch-mode power supply unit with adapters: • Input: 100 V – 240 V~, 50 Hz – 60 Hz • Output: 12 V= / 0.6 A	68 32 10 270 009
Ramp <b>seca 470</b>	470 0000 009
Accessories for integration via WiFi/LAN • External Interface module <b>seca 452</b> • Software <b>seca connect 103</b> • Barcode scanner (medical device)	452 0000 009  Will be provided in the course of integration projects Cannot be ordered from seca; for recommendations, see system instructions for use for <b>seca 103/452</b>

## 13 DISPOSING OF THE DEVICE

Do not dispose of the device in your household waste. The device must be properly disposed of as electronic scrap. Follow your respective national regulations. For more information, please contact seca Service or an authorized service partner.

## 14 WARRANTY

There is a two-year warranty period from delivery for defects attributable to poor materials or workmanship. All movable parts, e.g. batteries, cables, power supply units, rechargeable batteries etc. are exempt. Defects which come under the warranty will be repaired for the customer free of charge against proof of purchase. Additional claims cannot be considered. Costs of transport to and from seca are the responsibility of the customer if the device is located somewhere other than the customer's headquarters. In the event of transport damage, claims under warranty can only be made if the complete original packaging was used for transport and the device was secured and fastened in it according to its originally packaged condition. You should therefore keep all packaging parts.

The warranty will be voided if the device is opened by persons not expressly authorized by seca to do so.

In the event of a warranty issue, please contact your local seca office or the dealer from whom you ordered the product.

## 15 DECLARATION OF CONFORMITY

seca gmbh & co. kg hereby declares that the product complies with the terms of the applicable European directives and regulations. The unabridged declaration of conformity can be found at [www.seca.com](http://www.seca.com).

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