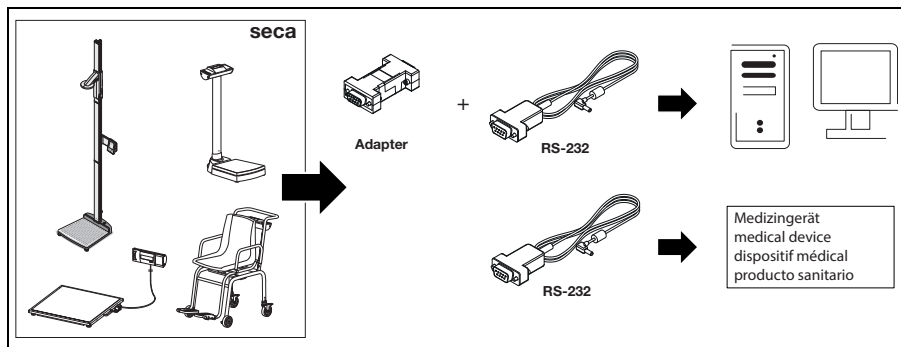
DE RS-232-SCHNITTSTELLE

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
EN RS232 INTERFACE

In addition to the standard functional scope, this scale is equipped with an RS-232 interface. The interface enables data transmission to a PC or dialysis system. The respectively appropriate interface cable is included in the scope of delivery:

Interface cable	Article number
For PC (incl. adapter with galvanic isolation)	08-04-08-005
For a dialysis system	08-06-16-167

Parameters for data transmission:

Baud rate	1200
Parity	Even
Data bits	8
Stop bits	1
Start bits	1
DTR/DSR	High / 1

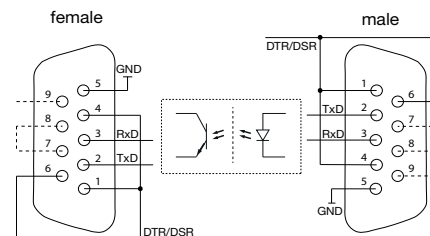CAUTION!

Hazard to patient, damage to device

Additional devices connected to medical electrical devices must provide evidence of compliance with the relevant IEC or ISO standards (e.g. IEC 60950 for data-processing devices). Furthermore, all configurations must comply with the requirements of standards for medical systems (see IEC 60601-1-1 or Section 16 of the 3rd edition of IEC 60601-1 respectively).

Anyone connecting additional devices to medical electrical devices is considered a system configurator and is therefore responsible for ensuring that the system complies with the requirements of standards for systems. Your attention is drawn to the fact that local laws take precedence over the above-mentioned requirements of standards. In the event of any queries, please contact your local specialist dealer or Technical Service.

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